


Measuring of frame wrap angle - ZTILT

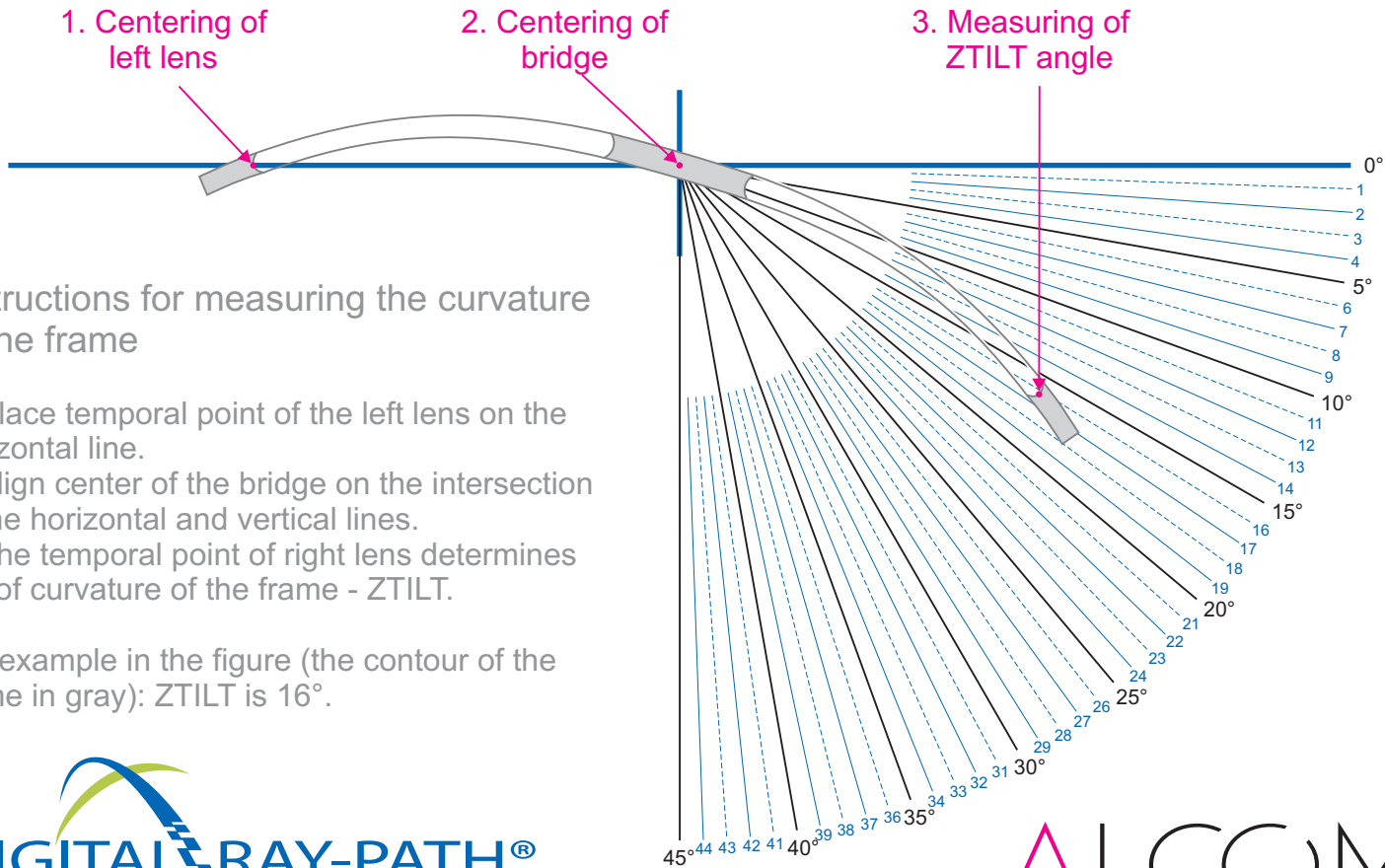

Instructions for measuring the curvature of the frame

1. Place temporal point of the left lens on the horizontal line.
2. Align center of the bridge on the intersection of the horizontal and vertical lines.
3. The temporal point of right lens determines the of curvature of the frame - ZTILT.

For example in the figure (the contour of the frame in gray): ZTILT is 16°.

DIGITAL RAY-PATH®
CALCULATION SOFTWARE

nanoHDVision
since 2009

ALCOM

20 YEARS OF
OPHTHALMIC TECHNOLOGY

www.alcom.info